

SS WHITE® Endodontic File System

ExactTaper H Treatment Protocol

SS White® ExactTaper™H shaping files have multiple tapers that ensure flexibility and cut dentin in specific canal zones with great efficiency, while reducing the potential for over-preparation. The ExactTaper™ finishing files insure the proper apical fit for the matching ExactTaper™ gutta percha points.

ExactTaper™H files are available in the same sizes, length and taper as the Dentsply® ProTaper® file system and Edge Endo® Edge Taper™ file system, which requires no change to current protocol or technique for greater ease of use, and offers the same predictable results.

- Identify the material makeup of any existing restorations that must be removed or penetrated to gain access, select the appropriate instrument for the task to gain access through the tooth.
- Use endodontic intra-canal lubricant and small hand files when manually scouting and securing canals
- Use NaOCl when mechanically pre-shaping or shaping canals
- Create glide-path using appropriate file system
- Use ExactTaper™H SX files in one or more passes to expand the glide path
- Use ExactTaper™H SX files to relocate the coronal aspect of a canal away from an external root concavity, eliminate triangles of dentin, or produce more shape as desired
- Irrigate, recapitulate with a hand file and re-irrigate after removing each rotary file
- Frequently clean and inspect the flutes of ExactTaper™H
- Use ExactTaper™H files in regions of the canal that have a confirmed, smooth and reproducible glide path
- Use ExactTaper™H files in a deliberate brushing motion, on the outstroke, to more optimally prepare canals that exhibit irregular cross sections and to facilitate apical file progression
- Use any ExactTaper™H file in one or more passes to safely achieve length
- Use the shaping files (S1, S2 and SX) with a brushing motion. Brushing motion
- Use the finishing files (F1-F5) in an 'in and out' action (not brushing)
- Withdraw the files once working length is established



simply better endodontics®

SS WHITE®


Shaping & Finishing Technique



Brush, Follow & Shape to Length
S1 & S2



Follow, Brush & Finish to Length
F1 & F2

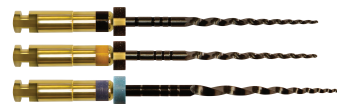

Expand File Shape as Desired
F3, F4, F5

Precautions

1. Before each use, inspect the file for damage or signs. Discard damaged files.
2. Maintain handpieces in accordance with the manufacturer's instructions.
3. Fully seat the file into the chuck mechanism to eliminate non-concentric operation.

Limitations of Liability

SS White® ExactTaper™H files may not be appropriate for all patients under all circumstances, so professional judgment and discretion should be exercised at all times. SS White® ExactTaper™H files should be used only by licensed dental professionals with appropriate levels of clinical experience and training. Caution and care should be used at all times. Treatment options and techniques outlined in this insert are suggestions only and may not be appropriate for all patients. Dentists must apply appropriate levels of force when using SS White® ExactTaper™H files. Excessive force can cause premature wear to the instruments. Use your professional judgment.

SS White® disclaims all liability and damages of any kind resulting from intentional or negligent conduct or otherwise that arise out of the use or misuse of any SS White® instruments or these instructions.