SS WHITE®

JAZZ®

POLISHERS

Directions for Use

1-step diamond polishing system for smoothing and polishing dental composites to the ultimate shine.

Indications:

Ultimate single-step polishing of all composite restorations.

Directions for use:

Trim and smooth restoration with **diamond** or **carbide finishing bur** to maximum smoothness.

- 1) **Polish** with Jazz Supreme beginning with firm pressure and higher speed. Maximum rpm: 15,000. Wipe restoration with cotton. Continue polishing with reduced pressure and speed with light touch to create desired shine.
- 2) For best results, polish wet.

Composition: All Supreme polishers contain a synthetic rubber matrix infused with diamond particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of stainless steel and are surface refined with gold flashing.

2-step diamond polishing system for smoothing and polishing dental composites to a high-luster.

Indications:

High-luster polishing of all composite restorations.

Directions for use:

- 1) Reduce and smooth with **medium grit (pink)** polisher.
- 2) Finish to high-luster with **fine grit (white)** polisher.

IMPORTANT: Clean restoration surface with gauze between polishing steps to remove polish residues.

Polish wet at 8,000 rpm. Maximum rpm: 15,000.

Composition:

All C2S polishers contain a synthetic rubber matrix infused with diamond particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of stainless steel.

1-step diamond polishing system for smoothing and polishing dental composites to a high-gloss.

Indications:

High-gloss single-step polishing of all composite restorations.

Directions for use:

- 1) **Polish** inished restoration until desired shine is achieved. Polish wet at 8,000 rpm. Maximum rpm: 15,000.

Disposable tool NOT for Reuse.

Composition:

All C1S polishers contain a synthetic rubber matrix infused with diamond particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of high performance polymer.

3-step polishing system for smoothing and polishing dental porcelain and metals to a high-luster.

Indications:

High-luster polishing of all porcelain & metal restorations.

Directions for use:

- 1) Reduce and trim with **coarse grit (green)** polisher.
- 2) Smooth with **medium grit (pink)** polisher.
- 3) Finish to high-luster with **fine grit (white)** polisher.

Polish wet or dry at 8,000 rpm. Maximum rpm: 15,000.

Composition:

All P3S polishers contain synthetic rubber, diamond grain in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of stainless steel.

2-step polishing system for smoothing and polishing dental porcelain and metals to a high-gloss.

Indications:

High-gloss polishing of all porcelain & metal restorations.

Directions for use:

- 1) Reduce and smooth with **medium grit (pink)** polisher.
- 2) Finish to final gloss with **fine grit (yellow)** polisher. Polish wet or dry at 8,000 rpm. Maximum rpm: 15,000.

Disposable tool NOT for Reuse.

Composition:

All P2S polishers contain a silicone matrix infused with silicon carbide particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of high quality polymer.

Warnings:

The product should not be used if a patient is known to be allergic to any of the ingredients of the polishing instruments. Avoid inhaling grinding dust.

Do not exceed maximum rotary speed.

Storage:

Storage temperature 2-28°C/36-82°F.

Do not expose to direct sunlight.

Keep out of the reach of children!

For use in dentistry only!

Cleaning Instructions:

- 1) Clean burs of visible debris. Automatic washing is preferable.
- 2) Autoclave on a short cycle at a temperature of less than 130°C.

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SS White® is a registered trademark of SS White Burs, Inc. Jazz® is a registered trademark of SS White Burs Inc. All rights reserved. Ver. 4 - 3/2012
SS White® ist eine geschützte Marke von SS White Burs, Inc.
Jazz® ist eine geschützte Marke von SS White Burs Inc. Alle Rechte vorbehalten.