V-CLEAN™ ENDODONTIC AGITATOR

Instructions For Use

V-Clean™ Endodontic Agitator is a medical-grade polymer hand instrument for cleaning the surfaces of a filed root canal. V-Clean™ is uniquely designed with multiple, flexible paddles to facilitate a dual-action scrubbing/scraping process that removes the smear layer, dislodges and removes debris from within the canal and concurrently provides an agitating action when used with irrigating solutions. In a University study, use of V-Clean™ reduced bacterial contamination within the root canal on average by 90.2% compared to use of irrigation alone.

1.

 Establish canal patency and prepare the root canal using SS White V-Taper™2 files, or reamers and files of choice.

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2.

 Rinse the prepared root canal with endodontic irrigating solution.

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3.

 Remove one V-Clean™ Endodontic Agitator from package.

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4.

 Insert the V-Clean™ Endodontic Agitator into the canal while manually rotating the brush in a circular, reciprocating motion.

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Instructions For Use

5.

Scrub the axial walls of the canal by manually rotating the V-Clean™ Agitator in a reciprocating, circular fashion to produce a dual-action scrubbing.

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6.

Continue manually rotating the V-Clean™ Endodontic Agitator as it is removed from the root canal.

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7.

For final cleaning, rinse and evacuate the root canal.

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8.

Discard the used V-Clean™ Endodontic Agitator properly and proceed to the root canal obturation.

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