
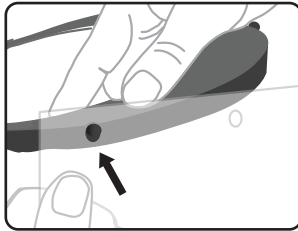
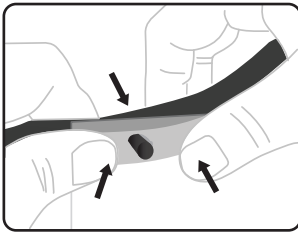
1. Remove items from box and place on a clean surface. Peel off protective cover from visor.

2. Ensure headset is positioned correctly with "top side" facing up. Align holes in visor with pins in headset (left to right).

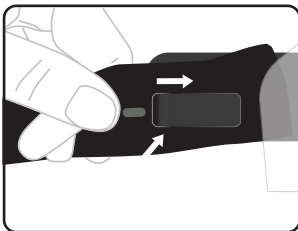3. Attach visor to headset by snap-fitting each prong to hole (left to right).

4. Thread headband band through tabs in headset.

5. Apply protective face shield to head and secure in place with headband.

Reusable Protective Face Shield

Personal protective equipment (PPE) designed to help protect your face, mouth and eyes from spray and splatter.

Product Features:

- Fast assembly and easy cleaning
- Adjustable Elastic band offers secure fit

DO NOT AUTOCLAVE

Each set includes one each of these component parts:

SAFETY TIPS

- Each headset should be cleaned between uses
- Parts should not be shared between persons without submitting for sterilization

Headset

Materials

Headband - Black ABS

Foam Liner - Closed Cell, Crosslined Polyethylene foam

Foam Insert - Closed cell, crosslinked Polyethylene foam

Usage

Multi-Use

Sterilization

PACKAGED NON-STERILE

Sterilization/Cleaning Instructions

Bleach (sodium hypochlorite)
0.5% (1:10 bleach: water), 2 mins
Soap/detergent + water
Cleaning of dirt only (not sterilizing)
air dry

Please Recycle

Visor

Dimensions

240mm x 317 mm

240mm x 360 mm

Materials

.51mm Thick PETG

Usage

Multi-Use

Sterilization

PACKAGED NON-STERILE

Sterilization/Cleaning Instructions

Isopropyl alcohol - 75-99% 5 minutes
Bleach (sodium hypochlorite) 0.5%
(1:10 bleach: water), 2 mins
Ethanol 70-80% maximum, 5 mins

Headband

Dimensions

Adjustable Elastic Band

Materials

Polyester Woven Elastic

Usage

Single-Use

Sterilization

PACKAGED NON-STERILE

Cleaning Instructions

Launder Wash / Standard Dry Cycle

Flexfit

The headband is designed for a adjustable fit. Re-position the headband for a more comfortable fit.

100% Satisfaction Guaranteed[†]

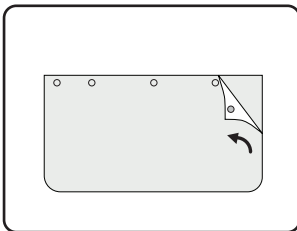
[†]SS White's 100% guarantee is on all of our products. If you are not completely satisfied with your purchase, we will issue a full refund of the purchase price.
©2020 SS White All Rights Reserved. SS White® is a registered trademarks of SS White Dental.

IFU-FaceShield

SS WHITE Dental®

Better Patient Outcomes
Improved Efficiency
Faster Practice Growth

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