JAZZ[®]

POLISHERS

Directions for Use

Risk if re-used - If product is used properly, after initial use, quality will be compromised after initial use.

1-step diamond polishing system for smoothing and polishing dental composites to the ultimate shine.

Indications:
Ultimate single-step polishing of all composite restorations.

Directions for use:
Trim and smooth restoration with **diamond or carbide finishing bur** to maximum smoothness.

- 1) **Polish** with Jazz Supreme beginning with firm pressure and higher speed. Maximum rpm: 15,000. Wipe restoration with cotton. Continue polishing with reduced pressure and speed with light touch to create desired shine.
- 2) For best results, polish wet.

Composition: All Supreme polishes contain a synthetic rubber matrix infused with diamond particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of stainless steel and are surface refined with gold flashing.

Risk if re-used - If product is used properly, after initial use, quality will be compromised after initial use.

2-step diamond polishing system for smoothing and polishing dental composites to a high-luster.

Indications:
High-luster polishing of all composite restorations.

Directions for use:
1) Reduce and smooth with **medium grit (pink)** polisher.
2) Finish to high-luster with **fine grit (white)** polisher.

IMPORTANT: Clean restoration surface with gauze between polishing steps to remove polish residues.
Polish wet at 8,000 rpm. Maximum rpm: 15,000.

Composition:
All C2S polishes contain a synthetic rubber matrix infused with diamond particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of stainless steel.

Risk if re-used - If product is used properly, after initial use, quality will be compromised after initial use.

1-step diamond polishing system for smoothing and polishing dental composites to a high-gloss.

Indications:
High-gloss single-step polishing of all composite restorations.

Directions for use:
1) **Polish** inished restoration until desired shine is achieved.
Polish wet at 8,000 rpm. Maximum rpm: 15,000.

Disposable tool NOT for Reuse.

Composition:
All C1S polishes contain a synthetic rubber matrix infused with diamond particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of high performance polymer.

Risk if re-used - If product is used properly, after initial use, quality will be compromised after initial use.

3-step polishing system for smoothing and polishing dental porcelain and metals to a high-luster.

Indications:
High-luster polishing of all porcelain & metal restorations.

Directions for use:
1) Reduce and trim with **coarse grit (green)** polisher.
2) Smooth with **medium grit (pink)** polisher.
3) Finish to high-luster with **fine grit (white)** polisher.
Polish wet or dry at 8,000 rpm. Maximum rpm: 15,000.

Composition:
All P3S polishes contain synthetic rubber, diamond grain in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of stainless steel.

Risk if re-used - If product is used properly, after initial use, quality will be compromised after initial use.

2-step polishing system for smoothing and polishing dental porcelain and metals to a high-gloss.

Indications:
High-gloss polishing of all porcelain & metal restorations.

Directions for use:
1) Reduce and smooth with **medium grit (pink)** polisher.
2) Finish to final gloss with **fine grit (yellow)** polisher.
Polish wet or dry at 8,000 rpm. Maximum rpm: 15,000.

Disposable tool NOT for Reuse.

Composition:
All P2S polishes contain a silicone matrix infused with silicon carbide particles in various sizes and pigments (mainly Titanium Dioxide). The shanks are made of high quality polymer.

Warnings:
The product should not be used if a patient is known to be allergic to any of the ingredients of the polishing instruments.
Avoid inhaling grinding dust.
Do not exceed maximum rotary speed.

Storage:
Storage temperature 2-28°C/36-82°F.
Do not expose to direct sunlight.
Keep out of the reach of children!
For use in dentistry only!

Cleaning Instructions:
1) Clean burs of visible debris. Automatic washing is preferable.
2) Vacuum autoclave 20 minutes 134°C.

Risk if re-used - If product is used properly, after initial use, quality will be compromised after initial use.

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