DESCRIPTION

SS White® EndoGuide® Burs are a unique series of eight burs for non-surgical root canal treatment. EndoGuide® Burs are specifically designed to maximize efficiency during endodontic exploration and access while conserving healthy tooth structure. The patented*, conical shaped micro-diameter tip of the EndoGuide® Burs acts as a self-centering guide to permit straight-line access to canals. EndoGuide® Burs blade design functions to effectively polish the dentin surface as it cuts, making visual dentin mapping easier and canal identification more precise. Two EndoGuide® kits address the specific endodontic needs of Anterior/Bicuspid teeth and Molar teeth. Individual EndoGuide® Burs are also available. (See Table A).

EndoGuide® Anterior/Bicuspid Kit
(For Endodontic Access and Exploration) #18052

Contains all instrumentation to create endodontic access through metal, porcelain and zirconia, featuring SS White® Great White® #2 metal cutting bur, Great White® Z Diamonds along with four EndoGuide® Burs ideally suited for locating and accessing single root canals in anterior/bicuspid teeth.

EndoGuide® Molar Kit
(For Endodontic Exploration) #18051

Contains seven EndoGuide® Burs designed to increase visibility and control during endodontic exploration in molars when locating hidden canals, navigating deeply calcified canals and troubleshooting between canals.

NOTE: Initial access and de-roofing in molars should be performed with either the Great White® #2 bur or Great White Z Diamonds supplied in the EndoGuide® Anterior/Biscupit Kit for Endodontic Access and Exploration, order #18052.

OPERATIONAL INSTRUCTIONS:

- EndoGuide® Burs are task specific; each bur is designed for a particular area of use during endodontic access and canal exploration.
- While EndoGuide® Burs are diminutive in size, they are very efficient cutters in both the vertical and lateral directions. To maximize cutting efficiency and avoid bur breakage, it is recommended that the clinician apply only enough pressure to guide the bur while allowing the instrument to progress through the substrate. When used for troughing procedures, a light sweeping motion is suggested.
- Excessive force to the instrument, especially in a lateral direction, can lead to bur damage.

INITIAL ACCESS THROUGH RESTORATIONS/TOOTH SURFACES:

- Identify the material makeup of any existing restorations that must be removed or penetrated to gain access.
- Once identified, select the appropriate instrument for the task. (See Table A)

NOTE: SS White® Complete Restoration Removal Kit (Order #18180) contains instrumentation specifically designed for efficient, non-traumatic removal of failed amalgams, composites, PFM crowns, and all-ceramic crowns, including zirconia and lithium disilicate ceramics.

ENDODONTIC ACCESS AND EXPLORATION:

EndoGuide® Burs are ideal for use when performing micro-endodontics using magnification-enhanced vision via a dental operating microscope or loupes. For best product performance, always match the EndoGuide® Bur to the task to be performed (See Table A).

Slow-speed EndoGuide®: Burs are those with SLRA (Surgical Length Right Angle) shanks and XLRA (Extra-Long Right Angle) shanks for use in standard slow-speed right-angle handpieces at slow-speed rpm ranges appropriate for endodontic procedures.

High-speed EndoGuide®: Burs are those with SLRA (Surgical Length Friction Grip) shanks for use in standard high-speed friction grip handpieces at high-speed rpm ranges appropriate for endodontic procedures.

PRECAUTIONS:

- Before each use, inspect the bur for damaged blades or signs of excessive wear. Discard worn burs.
- Maintain handpieces in accordance with the manufacturer's instructions.
- Fully seat the bur into the chuck mechanism to eliminate non-concentric operation.
- Do not exceed manufacturer's suggested speed recommendations.
- Clean and sterilize burs in accordance with the instructions provided by the manufacturer of your sterilization unit. Do not immerse carbide burs in cold sterilizing solutions or other strong oxidizing agents.

WARNINGS:

SS White® EndoGuide® Burs may not be appropriate for all patients under all circumstances, so professional judgment and discretion should be exercised at all times. SS White® EndoGuide® Burs should be used only by licensed dental professionals with appropriate levels of clinical experience and training. Caution and care should be used at all times. Treatment options and techniques outlined in this insert are suggestions only and may not be appropriate for all patients. Dentists must apply appropriate levels of force with SS White® EndoGuide® Burs. Excessive force can cause premature wear to the instruments. Use your professional judgment.

LIMITATION OF LIABILITY

SS White® disclaims all liability and damages of any kind resulting from intentional or negligent conduct or otherwise that arise out of the use or misuse of any SS White® instruments or these instructions.

1145 Towbin Avenue, Lakewood, NJ 08701 | 732-905-1100 / 800-535-2877
www.sswhitedental.com
U.S. Patent No. 6,257,889 B1
SS White® and Great White® are registered trademarks of SS White Burs, Inc. All rights reserved.
SS White® is a registered ISO 13485 Company. © 2010 SS White Burs, Inc.
Ver. 4 - 5/2021

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SS White® and Great White® are registered trademarks of SS White Burs, Inc. Todos los derechos reservados.
SS White® and Great White® are registered trademarks of SS White Burs, Inc. Tutti i diritti riservati.